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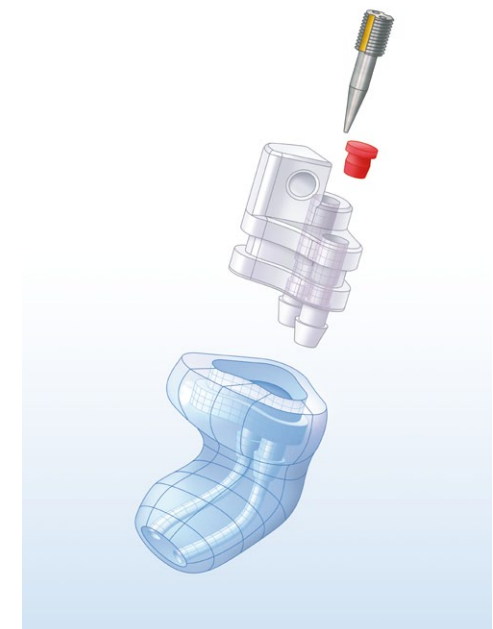
How do you effectively protect your hearing?

- Make a habit of it to always wear your Variphone hearing protection in areas with excessive noise.
- Never remove the blue/red caps.
- Never wear the Variphone (Soft) hearing protection without the filter module.
- The attenuation of your hearing protection has been set extremely precise to your personal noise load by means of the built-in filter module. Never modify the setting of the valve yourself!
- Have your hearing protection check regularly (annually) for its functionality. Checking whether the hearing protection is leak-proof and the attenuation is still correct is of great importance for its effectiveness.

How do you apply your hearing protection in your ear?

- Take the hearing protection from the supplied case with clean hands. The hearing protection is marked with a red or a blue cap. Red is for your right ear, blue is for your left ear.
- Take the right hearing protection at the grip of the module, so that the red cap now points at you.
- Place the hearing protection into your right ear. Subsequently turn the hearing protection a quarter backwards, so that the red cap points downwards. Subsequently, you can still push the hearing protection for a moment, should it not be fully in place.
- Take the left hearing protection into your left hand and apply it in the same manner.

How do you maintain your hearing protectors?

Correct maintenance can considerably extend the life-span of your hearing protection.

- Carefully keep your hearing protectors in the case supplied.
- Clean the hearing protectors after use with the supplied cloth. You can also clean the hearing protectors with lukewarm water. In that case it is best to remove the filter module before cleaning. Let the hearing protectors dry up a bit before you place this filter module back.
- If the auditory duct is clogged with earwax, you best use the supplied earwax bud to remove this earwax. NEVER use another object to remove the earwax.

Technical data

The Variphone Soft hearing protection complies with all criteria for the CE quality mark (PZT) under test number 09-1103-1 dd. 09/12/2011. Technical data according to DIN ISO Parts 1 and 2: see Table and Fig. 3.

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Mf = average attenuation | Sf = standard deviation
APVf = protective operation

The HearingCoach conservation program

It is recommended to have the Variphone Soft check annually on comfort and attenuation. All checks are carried out with a digital measuring device.

The Hearing Conservation Program is a preventive hearing care policy whereby the Variphone hearing protection is annually subjected to a new check. In this way, you remain protected to the maximum and the chances for noise damage are reduced to an absolute minimum.

Contact your distributor for more information about this product. You can contact us via info@variphone.nl or +31 (0)299-404449.

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